

# Archon 應用程式操作指南 (iOS / Android)

## 附錄3

### **ALIVE 固件升級方法 (使用 ALIVE)**

### **How to update Firmware (for ALIVE)**




# Archon App User Manual (iOS / Android)

手環有固件更新，請注意以下情況，如有彈出通知，按“好”；如沒有彈出通知出現，請選擇“設定”>”設備”，按“固件升級”

請將手環與手機保持近距離，升級需時約2分鐘

請將手環與手機保持近距離，等待固件升級

手環顯示  
Firmware  
Update

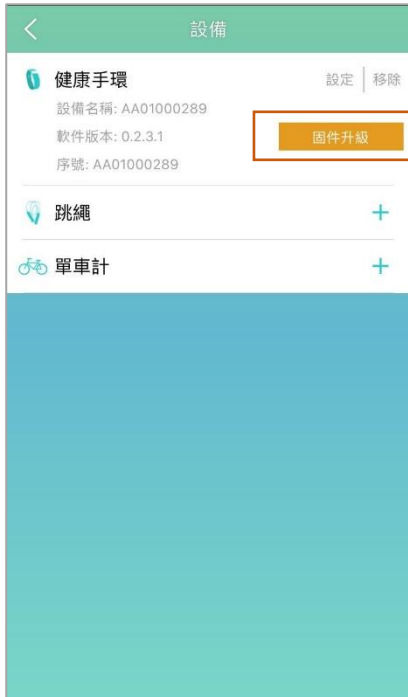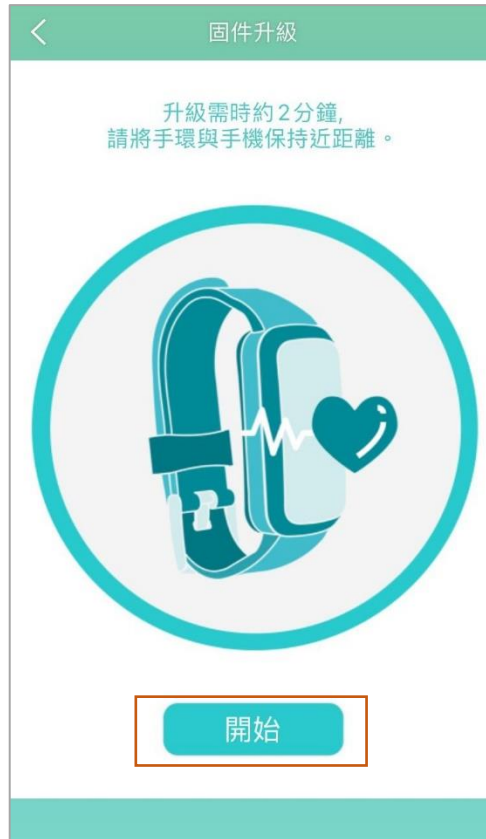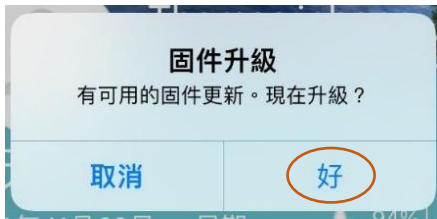





# Archon App User Manual (iOS / Android)

Pay attention to Firmware Upgrade: if there is a pop-up notification, press "OK"; if no pop-up notification appears, select "Settings">"Device", press "Firmware Available"

The process will take around 2 minutes. Please keep your mobile phone and ALIVE close to each other

Please keep your mobile phone and ALIVE close to each other and wait for the firmware upgrade

ALIVE display  
Firmware  
Update

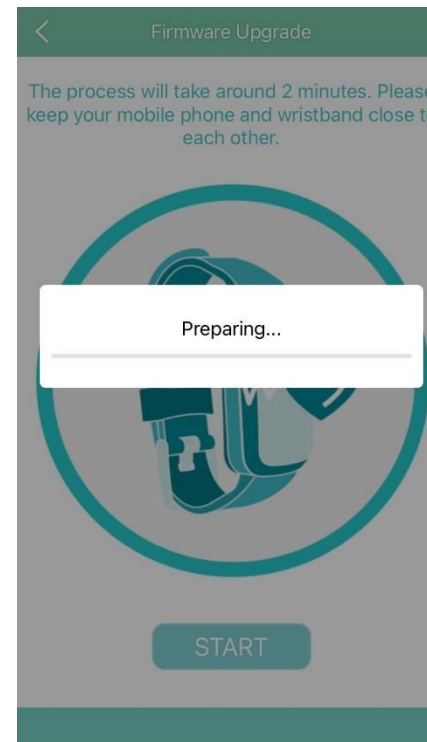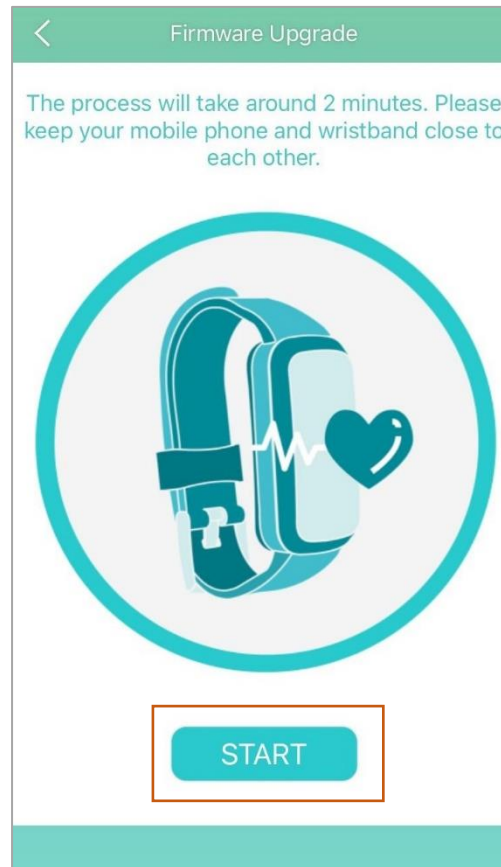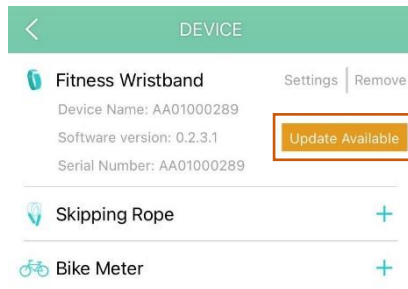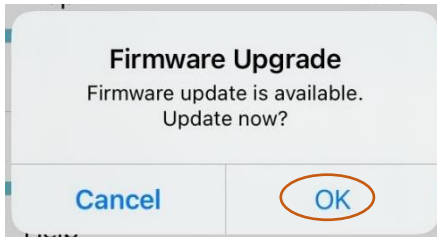


# Archon App User Manual (iOS / Android)

固件升級進行中

固件升級成功，按 "好"


如使用安卓手機，選擇 "設定">"設備同步"，按"同步"圖標，才會更新固件版本

確認固件版本  
0.2.9.1

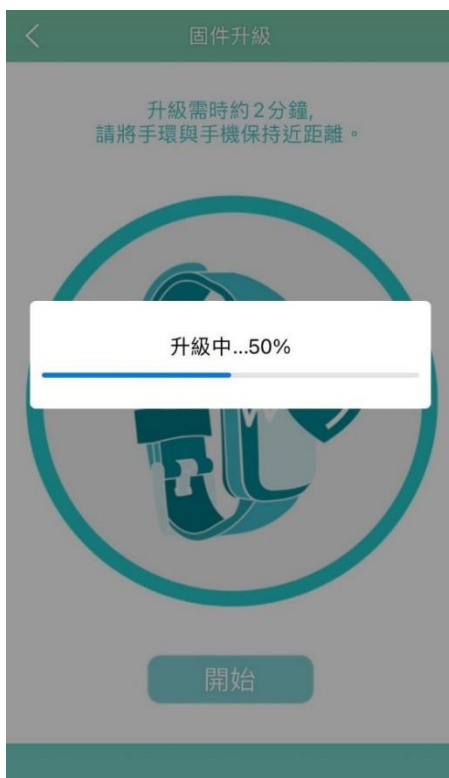


# Archon App User Manual (iOS / Android)


Firmware upgrade in progress

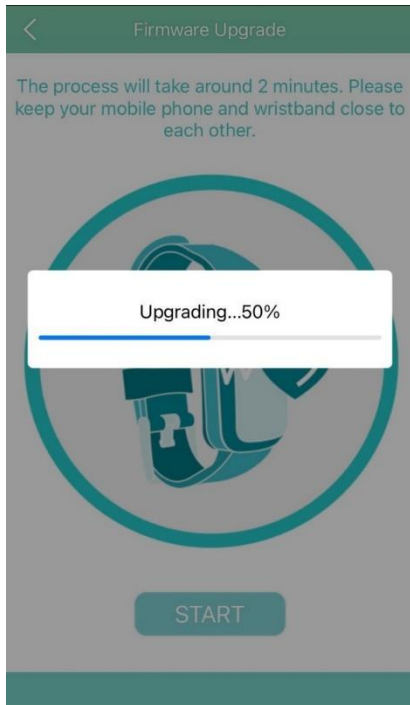


Firmware upgrade is successful, press "OK"

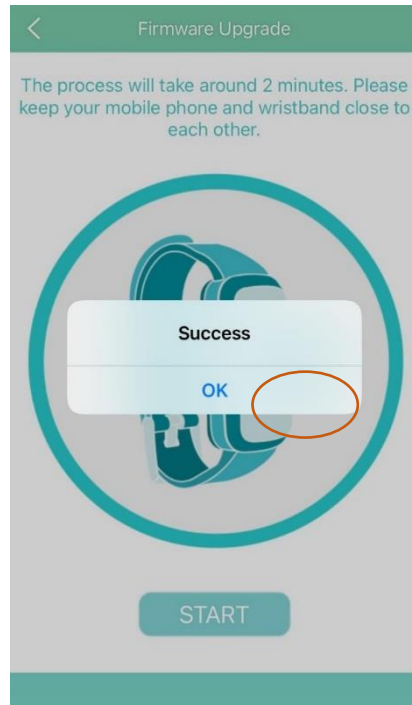

If use an Android phone, select "Settings" > "Synchronization" and click "Sync" icon to update the firmware version

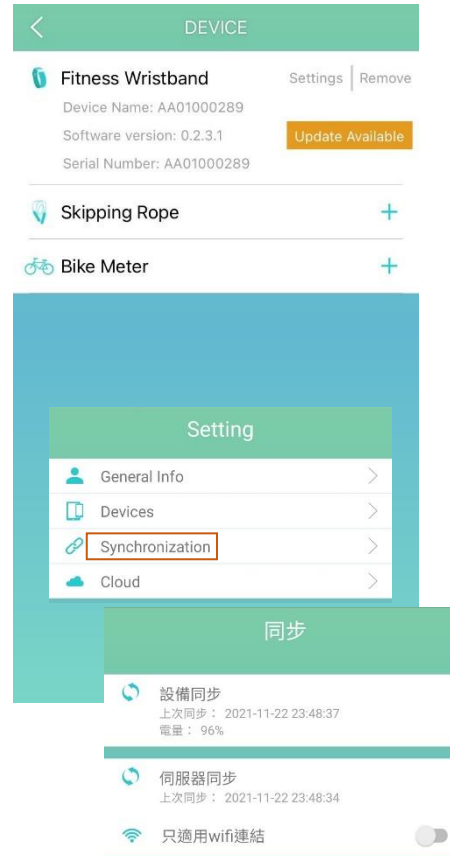

Confirm Firmware Version  
0.2.9.1

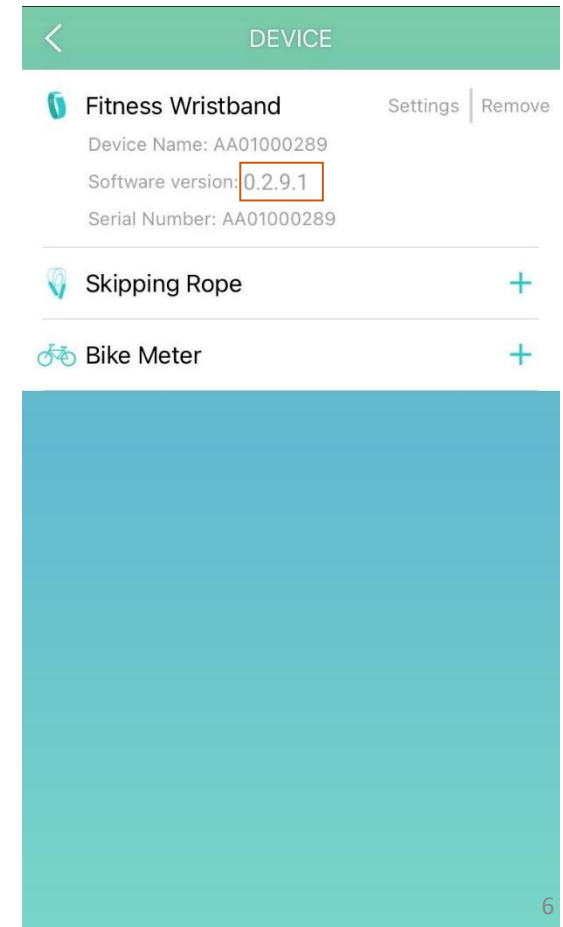

# Archon App User Manual (iOS / Android)

確認手環上已升級固件版本

1. 向下滑動




2. 按圖標



3. 按“設定”



4. 向下滑動

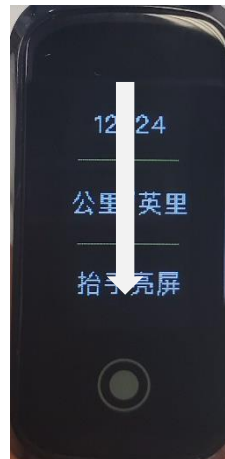

5. 按“設備信息”



6. 固件版本  
V0.2.9.1




# Archon App User Manual (iOS / Android)

## Confirm new Firmware version


1. Swipe down



2. Click Logo

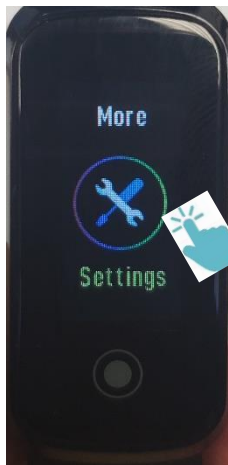


3. Click "Settings"

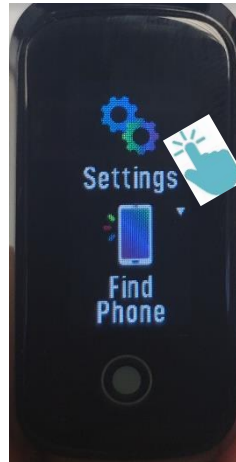

4. Swipe down



5. Click "About"

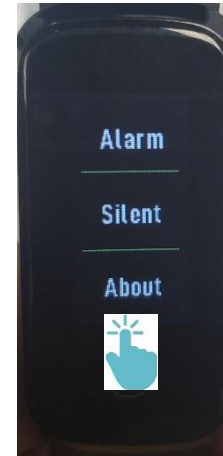

6. Firmware version  
V0.2.9.1

